

10%
Hydrogen
Peroxide

LumiSmile®
White
take-home

ON-THE-GO!

Dissolving Professional Teeth Whitening Strips

From the makers of LUMINEERS®

Mint Flavor



DIRECTIONS FOR USE

CLEAN HANDS

Wash hands thoroughly for 20 seconds with soap and warm water. Make sure hands are clean and dry.


OPEN SACHET

From the corner of the sachet, peel open and remove the strip.



APPLY UPPER STRIP

- Moisten the teeth, insides of the lips, and gums with the tongue and then apply the strip to the upper teeth.
- Center the strip and align the top edge of the strip with the gum line.
- Using a moist finger apply light pressure over the strip to adhere to the teeth.
- Fold the excess strip over the edge of the teeth so it adheres behind the teeth to keep the strip in place.
- Run your tongue over the newly applied strip to moisten.

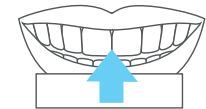


APPLY LOWER STRIP

- Repeat the process for the lower teeth.

WHITEN

- The Strip will dissolve slowly in approximately 15 minutes. Avoid eating and drinking until the strip is fully dissolved.
- There is no need for additional brushing or rinsing as the strip will dissolve completely.

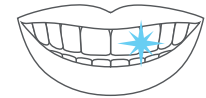

For more information, call 800-433-6628 or visit oralprocare.com

Store in a cool, dry place.

Keep out of reach of young children. Product is not intended for use in children under the age of 12.



©2021 Den-Mat Holdings, LLC. All rights reserved.
Distributed by DenMat, Lompoc, CA 93436 USA
753714100 R:0 02/21SN